Philips AED Inspection Form

Stamped by Philips on battery

Philips AED Inspection Form		*LIFEGUARD			
AED Serial Number: Located on back of device	Adult Pads Expiration Date:	Life Safety, Life Saving & First Aid Products			
Premise Location:	Pediatric Pads Expiration Date:	Workplace Safety Linearung			
Battery Install Date: 4 year life once installed	Spare Pads Expiration Date:	 402-898-4141 877-868-4141			
Battery Expiraton Date:	Spare Battery Install Before Date:	www.lifeguardmdusa.com			
Battery Install Before Date:	<u> </u>	info@lifeguardmdusa.com			

Check the following indicated inspected with a check mark in box.	<u>Date</u>											
Is the green ready light blinking indicating that the unit is ready to use?												
Is the unit clean, undamaged, and free of cracks or visible damage?												
Verify that defibrillation pads are within expiration date. Replace if expired.												
Verify electrodes are connected to the unit and sealed.												
Are all cables, if visible, free of cracks, cuts and exposed or broken wires?												
Verify battery is within expiration date. Replace if expired.												
Check Response Bag for gloves, mask, scissors, razor, wipes & gauze.												
Alarm on AED Cabinet operates correctly. Change battery yearly.												
Inspected by:												
Report any changes to LifeGuard MD 877- 868-4141												